



Infa-Lab, Inc.

11 Wall Street; Rockaway, NJ 07866
PO Box 691; Rockaway, NJ 07866
973-625-2265 Telephone:
800-247-7527 Toll Free:
973-625-5641 Facsimile:

Brace Gard™ Bracewax is the ultimate in bracket protection! It is simply pinched over the offending bracket, pushing material under and behind the bracket, forming a gripping configuration (mechanical retention). Special food grade silicone wax will not melt or become hard and crumbly. Brace Gard™ is transparent for a pleasing appearance and remains pliable allowing removal and re-application.

Instructions for use:

1. Pinch BRACE GARD® material over dry bracket, locking material behind edges and undercuts.
2. Finish by smoothing with a wet finger.

IMPORTANT:

Material and bracket must be dry.

