



Infa-Lab, Inc.

11 Wall Street; Rockaway, NJ 07866

PO Box 691; Rockaway, NJ 07866

973-625-2265 Telephone:

800-247-7527 Toll Free:

973-625-5641 Facsimile:

At the first sign of irritation apply Brace Relief™ Gel to minimize discomfort from brackets and wires rubbing on the cheeks an/or gum area during orthodontic treatment.

Instructions for use of Brace Relief™

1. Wash hands with soap and water.
2. Pull cheek back and identify irritated area.
3. Place a small amount of Brace Relief™ on finger or cotton swab and apply to irritated area.