


Use Brace Relief™ Gel as a pain relieving lubricant to numb the periodontal membrane when applying separator elastics (spacers), ESSIX™* appliances or Invisalign®** appliances.

Instructions for use with separating elastics:

1. Dispense a small amount of Magic Touch Brace Relief™ Gel on a mixing pad.
2. Dip separating elastics in Brace Relief™ Gel then place separating elastic between teeth to be separated.

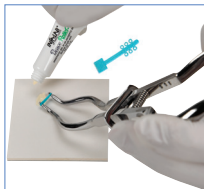

Instructions for use with Essix and Invisalign® Appliances:

1. Dispense a thin coating of Magic Touch Brace Relief™ Gel along the indenture (imprint of the teeth).
2. Place appliance in patient's mouth and remove any excess gel that is expressed at the gingival aspect of the aligner.

