

Infa-Lab, Inc.

11 Wall Street; Rockaway, NJ 07866
PO Box 691; Rockaway, NJ 07866
973-625-2265 Telephone:
800-247-7527 Toll Free:
973-625-5641 Facsimile:

Infa-Lab Magic Touch™ Brace Gard® Lip Protectors are specially designed for orthodontic patients to provide a comfortable, easily installed shield that protects their lips from abrasion when playing musical instruments and participating in sporting activities.

To Professionally Custom Fit:

1. Starting at the midline apply slight pressure to slide lip protector wire guides onto archwire.
 2. Continue working around the arch applying pressure to wire guides to fit onto archwire.
- * If wire guide (gripper) aligns to a bracket, simply remove with scissor or distal end cutter. Continue working around the arch until fully placed.

